


# Colostomy bag instruction

Two System

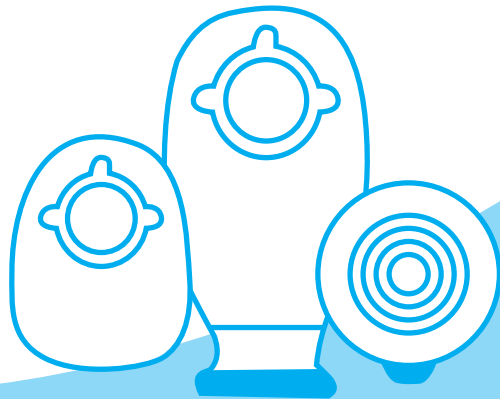

## Two System Colostomy Bag

1. Use the water to clean the skin around stoma.

2. A hole have been advanced open at the, flange the sufferers can cut a hole at the flange at the basis of the kind of stoma.



3. Remove the protection paper at the flange.



4. Put the flange which have been cut well aimed at the stoma and stick on uncitly, press it down lightly by using the finger, make the flange stick on well. Notice: the two microtias which include the hole and the sash at the parallel direction.



5. Blow little air into the bag before fixing the bag in order to prevent the wall of the pocket being sticked on .then hold the cutting sleeve aimed at the wedge seat using the two hands, from the bottom, the finger press it carefully along the cutting sleeve, make wedge seat and the cutting sleeve solid absolutely.



6. When have to change the bag put one finger to press on flange, another finger to press the middle, little protruding of the cutting sleeve and pult forth, unbolt the bag.



7. Use one finger to press the skin, another finger to tear out flange carefully when have to change the flange.



8. Use clip to clamp the end of bag, agraiffe type must be press the end of bag and turn forth 2 circum ference then join the agraiffe before use the drainable bag.



9. Open the end of the bag when giving out stool, make the end of the bag aim at the urinal, and then wash the end of the bag as the picture, make the end of the bag smoothly, then can reuse.




## Notice

1. The sufferer of the intestines open can use this product to incept the ordure only.

2. The "flange" to mix this product can use 3 - 5 days in normal condition, it can affect the effect of the affixing if use delay, it is not good to the skin .

3. If the bag full of air, make it exhausted by using the pinhole in the upper end of the bag. suggest use activated carbon cassette, it can wipe off the peculiar smell.

4. The chassis and the bag which being exchanged can not be used againg.

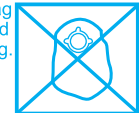

5. The flange and the bag which can not be washed into the toilet or cloaca, it can not be discarded at will, pay attention to the protection of the environment.



6. The skin uses to the gum allergy carefully.